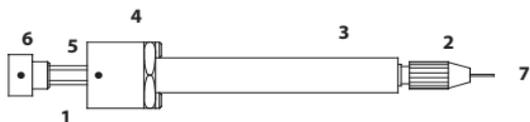
- EN**
7 **Flexipet® Adjustable Handle Set**
Instructions for Use
- BG**
9 **Комплект с регулируема дръжка Flexipet®**
Инструкции за употреба
- CS**
11 **Souprava nastavitelné rukojeti Flexipet®**
Návod k použití
- DA**
13 **Flexipet® justerbart håndtagssæt**
Brugsanvisning
- DE**
15 **Flexipet® Einstellbares Griff-Set**
Gebrauchsanweisung
- EL**
17 **Σετ ρυθμιζόμενης λαβής Flexipet®**
Οδηγίες χρήσης
- ES**
19 **Equipo de mango ajustable Flexipet®**
Instrucciones de uso
- ET**
21 **Flexipet® reguleeritava käepideme komplekt**
Kasutusjuhend
- FR**
23 **Set de poignée réglable Flexipet®**
Mode d'emploi
- HR**
25 **Komplet podesive drške Flexipet®**
Upute za uporabu
- HU**
27 **Flexipet® állítható fogantyúkészlet**
Használati utasítás
- IT**
29 **Set con impugnatura regolabile Flexipet®**
Istruzioni per l'uso

T _ K - M P H _ R E V Z

- LT**
31 „Flexipet®“ reguliuojamosios rankenos rinkinys
Naudojimo instrukcija
- LV**
33 Flexipet® regulējama roktura komplekts
Lietošanas norādījumi
- NL**
35 Flexipet® verstelbare handgreepset
Gebruiksaanwijzing
- NO**
37 Flexipet® sett med justerbart håndtak
Bruksanvisning
- PL**
39 Zestaw regulowanego uchwytu Flexipet®
Instrukcja użycia
- PT**
41 Conjunto de punho ajustável Flexipet®
Instruções de utilização
- RO**
43 Flexipet® Set cu mâner ajustabil
Instrucțiuni de utilizare
- RU**
45 Комплект регулируемой рукоятки Flexipet®
Инструкция по применению
- SK**
47 Súprava nastaviteľnej rúčky Flexipet®
Návod na použitie
- SL**
49 Komplet nastavljivega držala Flexipet®
Navodila za uporabo
- SV**
51 Flexipet® justerbart handtagsset
Bruksanvisning
- TR**
53 Flexipet® Ayarlanabilir Sap Seti
Kullanma Talimatı

1

EN

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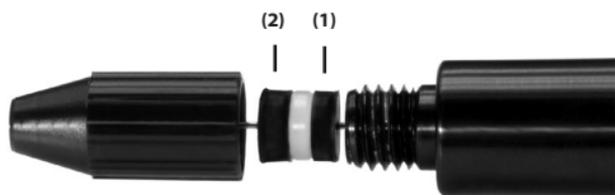
5

FLEXIPET® ADJUSTABLE HANDLE SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Flexipet® Adjustable Handle Set is used with the Cook Flexipet denuding and manipulation pipettes in the manipulation of oocytes, embryos, and biopsy cells. This device is intended for use by clinicians or technicians for women undergoing in vitro fertilization procedures. The adjustable handle accepts all sizes of Flexipet pipettes and the aspiration volume can be adjusted. The target population for the Flexipet Adjustable Handle Set includes women undergoing IVF procedures.

INTENDED USE

Used with Flexipet pipettes for denuding of oocytes and manipulation of oocytes, embryos, and biopsy cells.

INDICATIONS FOR USE

This device is used with the Flexipet pipettes for manipulation of oocytes, embryos, and biopsy cells during the IVF procedures for patients with infertility or difficulty in achieving a natural pregnancy.

CLINICAL BENEFITS

Successful holding and facilitating the use of Flexipet pipettes during denuding and manipulation of oocytes, embryos, and biopsy cells.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

None known

HOW SUPPLIED

Supplied non-sterile in peel-open packages with a set of replacement o-rings. Reusable. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

INSTRUCTIONS FOR USE

Prior to each use, ensure plunger operation is smooth and consistent, and Flexipet pipette is properly secured to the handle.

Adjusting Aspiration Volume (Fig.2)

- The device is not intended for delivering exact measurements. Reference marks are provided as guidance to adjust aspiration volume. To change volume, twist plunger to the desired setting on the plunger scale, until the reference mark on the finger pad is opposite the reference mark on the spring housing.
- Rotate the plunger clockwise to decrease aspiration volume.
- Rotate the plunger counterclockwise to increase aspiration volume.

Installing Flexipets

1. Loosen the collet by turning counterclockwise at least one full rotation.
2. Slide a Flexipet onto the plunger tip and gently advance into the collet until firmly seated. Tighten the collet.

IMPORTANT: Over-tightening the collet can compress the o-rings and restrict plunger movement. If the plunger fails to move freely, loosen the collet until the desired movement is achieved.

Aspiration Technique

1. Depress the plunger until the finger pad contacts the spring housing, immerse the Flexipet tip into the culture media, and slowly release the plunger.
2. Expel media by depressing the plunger until the finger pad is completely depressed [blow out] (**Fig. 3**). To avoid air bubbles, stop the plunger before it is completely depressed. The small amount of medium remaining in the extreme tip can be touched off.

O-Rings Replacement

O-rings should be replaced if plunger operation is not smooth and consistent, leakage occurs or if the Flexipet cannot be properly secured to the handle.

IMPORTANT: To ensure proper operation, the o-rings must be installed as depicted in **Fig. 4** and **Fig. 5**, with the smaller aperture o-ring (1) installed first. Also, the second o-ring (2) must be installed with the larger aperture positioned toward the collet so the Flexipet will fully insert into the o-ring (**Fig. 5**). A hex key is provided to push the o-rings out of the collet during replacement, if needed.

A handle o-ring replacement set (K-FRO-1000) may be ordered, consisting of one small aperture o-ring, one large aperture o-ring, spacer and collet.

Handle Replacement

The handle should be replaced when the handle and installed pipette no longer aspirate oocytes or embryos and o-ring replacement is ineffective.

DISPOSAL OF DEVICES

Dispose of all used or inoperative products and associated packaging per local medical waste disposal procedures.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

SERIOUS INCIDENT REPORTING

If any serious incident has occurred in relation to the device, incident should be reported to Cook Medical and shall also be reported to the competent authority where the device was used.

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© COOK 2021
WILLIAM A. COOK AUSTRALIA Pty Ltd
U.S.A.
COOK MEDICAL EUROPE Ltd